

## Keeler BIO Breath Shield Installation Instructions

Ref: AX17448

### Installation on Keeler BIO units:



#### Step One

To install the Keeler BIO breath shield, unsnap the front of the headband foam liner.



#### Step Two

Once the holes are exposed, align the holes of the breath shield with the headband snaps.



#### Step Three

After the holes are aligned, snap the headband foam liner into the original closed position.


*\*We recommend the Denser 1" hook and loop velcro tape with sticky back for adhering the shield to the Heine BIO.*

### Installation on the Heine 500™ unit:



#### Step One

Remove the headband comfort pad by peeling back the velcro. Keep the comfort pad to the side.

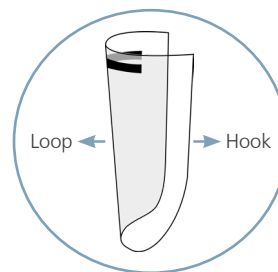

#### Step Two

Center and apply a 7 inch strip of 1" wide velcro tape onto both sides of the top portion of the shield. The velcro should cover the holes.\*

*Note: The patient facing portion of the shield should have the "fuzzy side" (loop) of the velcro tape. The user side of the shield should have the "rough side" (hook) of the velcro.*



#### Step Three

With velcro attached to the shield, place the side with the loop against already existing hook velcro inside the BIO headband.



#### Step Four

Press comfort pad against the hook velcro already placed onto shield.